


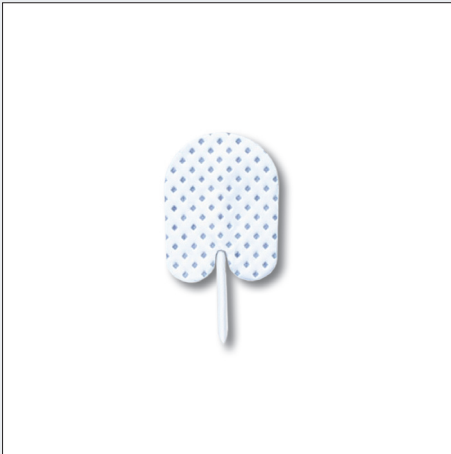

Key features / Benefits

- Small size
- Special "duck foot" design ensures excellent adhesion
- Flexible soft material
- Repositionable
- Ag/AgCl sensor for best trace quality
- Available in different leadwire configurations

Ambu[®] Neuroline 700

Ambu Neuroline 700 is a small size, multiple purpose neurology electrode that can be used for all neurological examinations where the electrode is placed outside the hair zone. Ambu Neuroline solid gel electrode is self-adhesive, and it is easy to reposition. Ambu Neuroline 700 is ideal for examinations of the face, infants and children.

The solid gel electrode is made of a very soft and flexible material, which ensures patient comfort and trace quality even during sleep studies. The 700 electrode is available with white or colour-coded lead wires in different lengths.

Recommended applications

- Surface Electromyography (EMG)
- Nerve Conduction Studies (NCS)
- Electronystagmography (ENG)
- Electro-Oculography (EOG)
- Polysomnography (PSG)
- Evoked Potentials (EP)
- Reference electrode

DIMENSIONS

Electrode size (max. L x W or diameter in mm)	20x15
Skin contact size	
(max. L x W or diameter in mm)	20x15
Adhesive area (in mm ²)	263
Height excluding connector/wire (in mm)	1

SENSOR

Sensor material	Silver/silver chloride
Gel system	Solid gel
Sensor area (in mm ²)	263
Gel area/measuring area (in mm ²)	263



ENVIRONMENT

PVC-free electrode	No
Latex-free electrode	Yes
PVC-free lead wire	No
Latex-free lead wire	Yes
PVC-free packaging	Yes

MATERIALS - PACKAGING

Pouches, inner layer	Polyethylene (PE)
Pouches, centre layer	Aluminium (Al)
Pouches, outer layer	Paper
Boxes	Cardboard

MATERIALS - ELECTRODE

Bio-compatible	Yes
Sensor	Silver/silver chloride (Ag/AgCl)
Outer carrier	Carbon filled polyvinyl chloride
Outer adhesive	Solid gel
Upper part	Non-woven polypropylene
Lead-wire core	Flexible copper
Lead-wire insulation	Polyvinyl chloride (PVC)
Lead-wire connector	Brass/polypropylene (PP)
Liner	Siliconized polyester film



K = 1,5 mm connector



J = 2 mm connector



SC = 0.7 mm connector

AVAILABLE CONFIGURATIONS

Item No.	Lead wire length		Colour	Shelf life in months		Packaging		
	cm	inch		(unopened/open pouches)		pouch	box	carton
700 01-K/12	10	4"	□	18	1	12	240	1920
700 01-SC/12	10	4"	□	18	1	12	240	1920
700 02-K/12	20	8"	□	18	1	12	240	1920
700 02-SC/12	20	8"	□	18	1	12	240	1920
700 05-J/12	50	20"	□	18	1	12	240	1920
700 05-K/12	50	20"	□	18	1	12	240	1920
700 08-K/12	80	32"	□	18	1	12	120	960
700 10-J/12	100	40"	□	18	1	12	120	960
700 10-K/12	100	40"	□	18	1	12	120	960
700 10-K/C/12	100	40"	■ ■ ■ □	18	1	12	120	960
700 15-J/12	150	60"	□	18	1	12	120	960
700 15-K/12	150	60"	□	18	1	12	120	960
700 15-K/C/12	150	60"	■ ■ ■ □	18	1	12	120	960
700 20-K/C/4	200	80"	■ ■ ■ □	18	1	4	480	960

DIRECTIONS

Prepare the skin using an abrasive. Degrease using alcohol swab. Peel the electrode from its protective backing and apply. The electrode can be repositioned several times for better results..

PRECAUTIONS

Single patient use only. For monitoring and recording biopotential on the skin.



Storage temperature

Ambu A/S

Baltorpbakken 13
DK-2750 Ballerup
Denmark
T +45 72 25 20 00
F +45 72 25 20 50
www.ambu.com

NEUROLOGY