




Home care masks and accessories

A solution for every patient

PHILIPS

RESPIRONICS

Which mask for which patient?

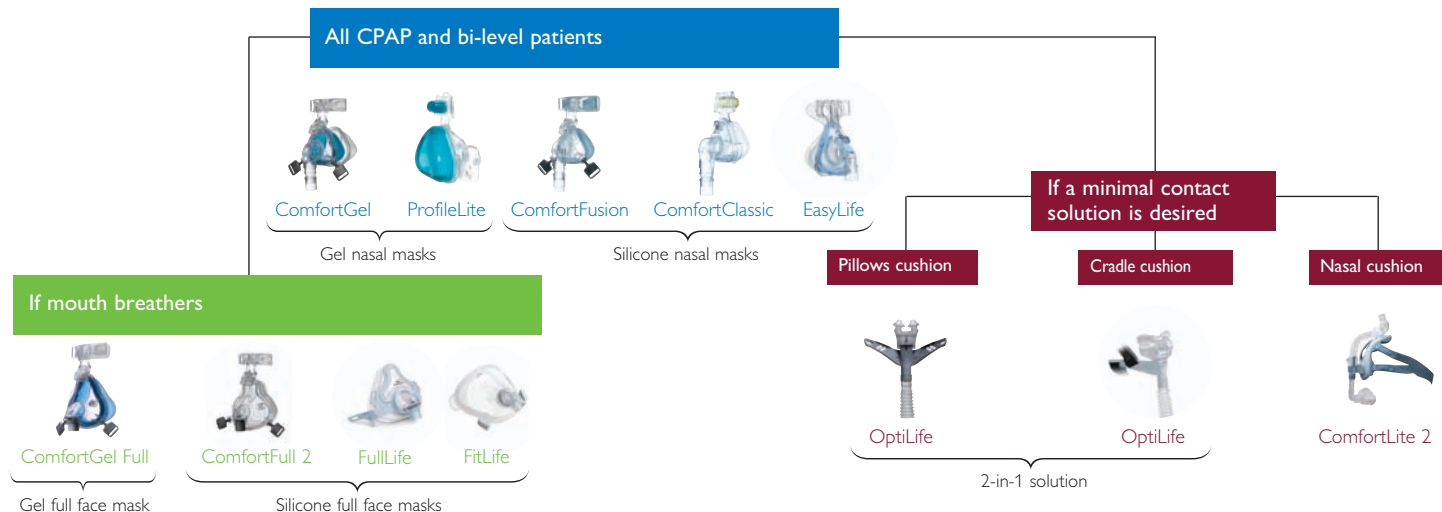

Nasal masks

Full face masks

Minimal contact masks

Accessories

Mask fitting

Devices

Patient interface line overview

ComfortGel



ProfileLite



ComfortFusion



ComfortClassic



EasyLife



Nasal masks

Sizes	P, S, M, L	P, S, SM, M, ML, L, LN, Child	S, M, FitPack (S/M) DuoPacks (S/S, M/M)	S, M	S, M, MW, L FitPacks (S/M and MW/L) DuoPacks (S, M, MW, L)
SE version (no exhalation port)	Yes	Yes	Yes	No	Yes
Replaceable cushion	Yes	No	Yes	No	Yes

ComfortGel Full



ComfortFull 2



FullLife



FitLife



ComfortLite 2

OptiLife
2-in-1 solution**Full face masks**

Sizes	S, M, L	S, M, L	S, M, L	S, L
SE version (no exhalation port)	Yes	Yes	Yes	Yes
Replaceable cushion	Yes	No	Yes	No

Minimal contact masks

P, S, M, L	P, S, M, L	S, M, L, LN
No	No	No
Yes	Yes	Yes

Nasal masks


Full face masks

Minimal contact masks

Accessories

Mask fitting

Devices

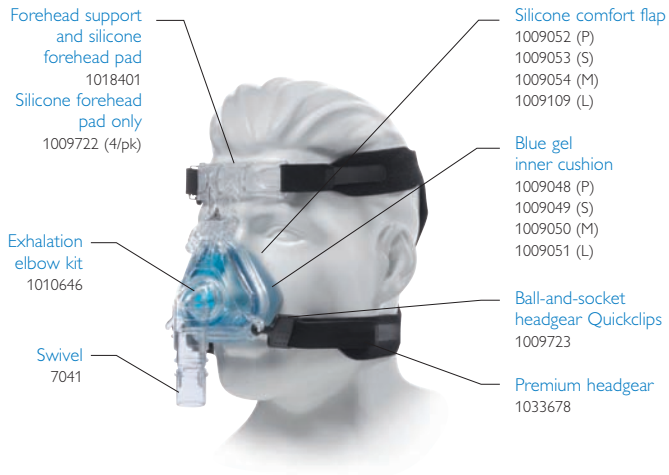


ComfortGel

The unique gel technology to optimize comfort and drive long-term therapy



Without exhalation port

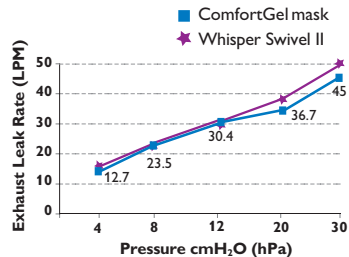


Size	P	S	M	L
With exhalation	1009044	1009045	1009046	1009047
Without exhalation	1010642	1010643	1010644	1010645

ProfileLite

The original gel mask without silicone and fewer adjustment features



Without exhalation port

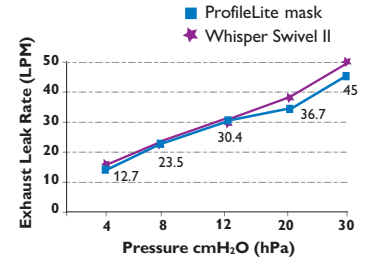

Size	P	S	MS	M	MW	L	LN	PLite Youth Child
With exhalation	1004113	1004114	1004115	1004116	1004117	1004118	1004119	1054707
Without exhalation	1006314	1006315	1006316	1006317	1006318	1006319	1006320	

Gel retrofit
head spacer
1004292 (S)
1004291 (L)
1008869 (XL)

Exhalation
elbow kit
1004395

Swivel
7041


Port cap
152033 (10/pk)

Deluxe headgear
1002801

Small child
ProfileLite
(over one year of age)
1008829

Profile Lite Youth
(7 years or older)
1054707

Nasal masks

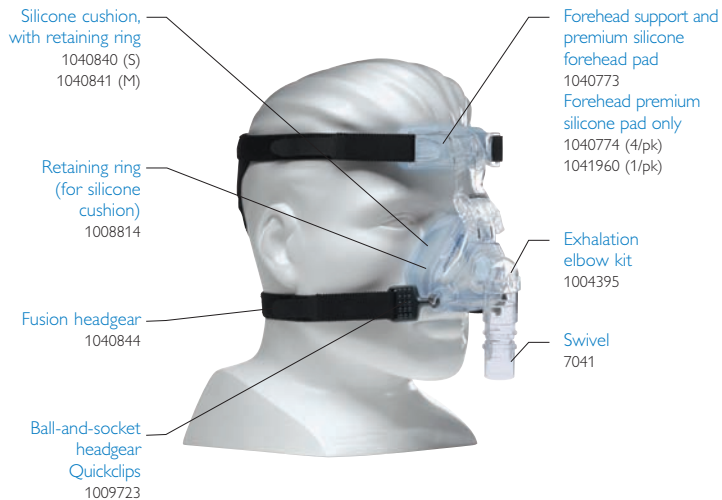


ComfortFusion


The most effective replacement mask for silicone nasal mask users



Without exhalation port

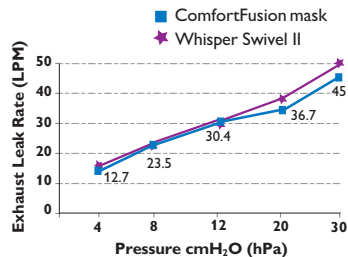

Size	S	M	FitPack S and M	DuoPack S and S	DuoPack M and M
With exhalation	1042211	1042212	1040838	1042213	1042214
Without exhalation	1042215	1042216			

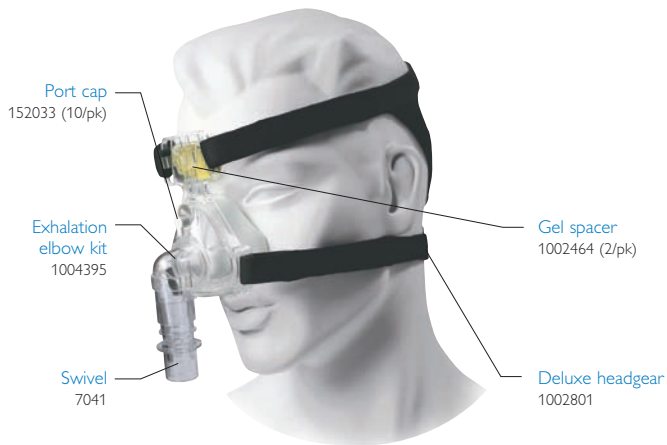


ComfortClassic

The simplest silicone mask with fewest adjustments



302209

Mask swivel without exhalation port to be connected when no port on the mask is required.

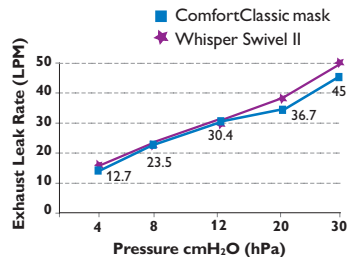

Size	S	M
With exhalation	1007967	1007968


EasyLife headgear
1050095

Cushion sets*
1050091 (S)
1050092 (M)
1050093 (MW)
1050094 (L)




EasyLife

The new self-adjusting premium silicone nasal mask with AutoSeal technology

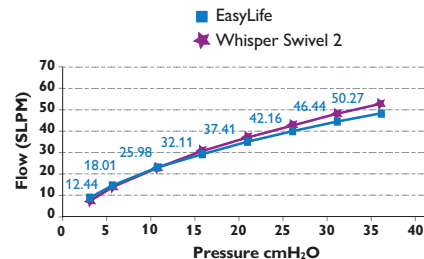

Size	S	M	MW	L
With exhalation	1050021	1050022	1050023	1050024
Without exhalation	1050031	1050032	1050033	1050034
DuoPack (extra cushion)	1050026	1050027	1050028	1050029
FitPack (mask with headgear and cushions S and M)				1050088
FitPack (mask with headgear and cushions MW and L)				1050089

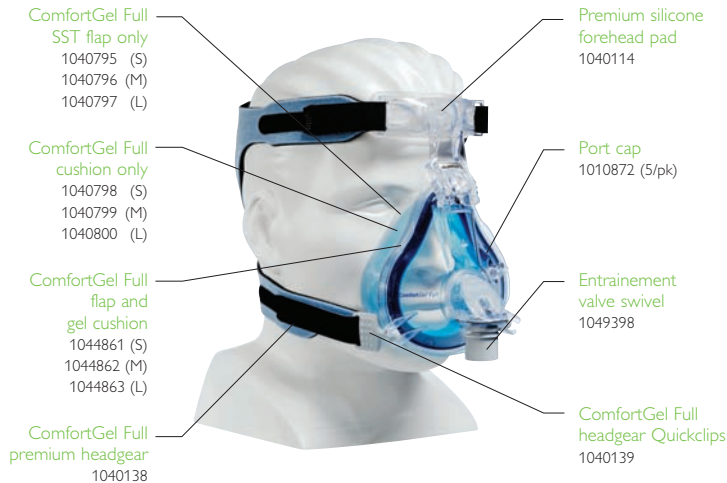


Full face masks

ComfortGel Full


The recognized gel technology now available for mouth breathers.
Immediate comfort and long-term compliance.



Without exhalation port

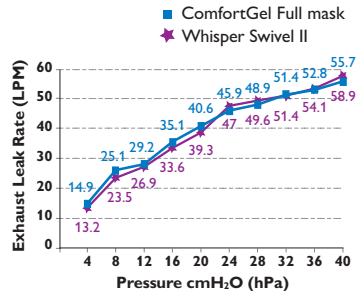

Size	S	M	L
With exhalation	1040140	1040141	1040142
Without exhalation	1043550	1043551	1043552

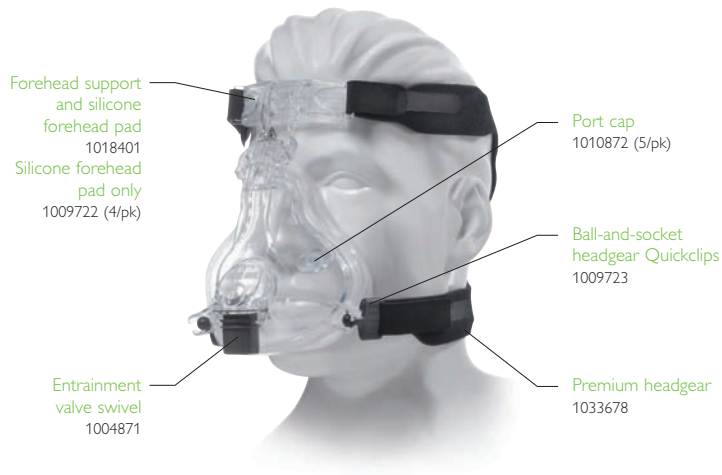


ComfortFull 2

The basic silicone full face mask for mouth breathers




Without exhalation port

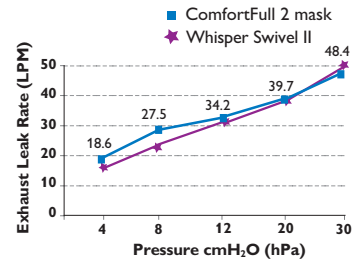

Size	S	M	L
With exhalation	1004881	1004873	1004951
Without exhalation	1023747	1024261	1024262

FitLife

A total face mask that provides comfort for hard-to-fit patients

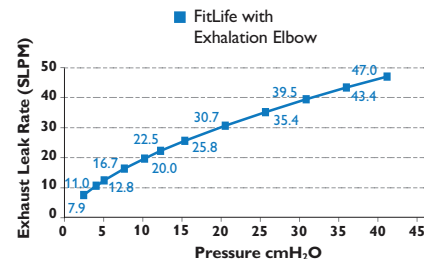

Size	S	L
With exhalation	1060803	1060804
Without exhalation	1061711	1061712


Full face masks

FullLife

A minimal-contact, full-face mask with clear line of vision




Without exhalation port

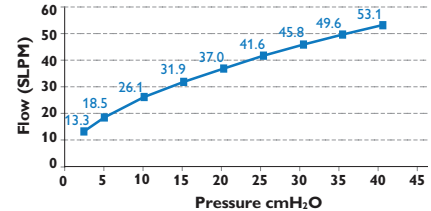

Size	S	M	L
With exhalation	1047939	1047940	1047941
Without exhalation	1047943	1047944	1047995
DuoPack (extra cushion, same size)	1052167	1052168	1052169
FitPack (1 mask with 3 cushion sizes)	1047942		

Full face masks

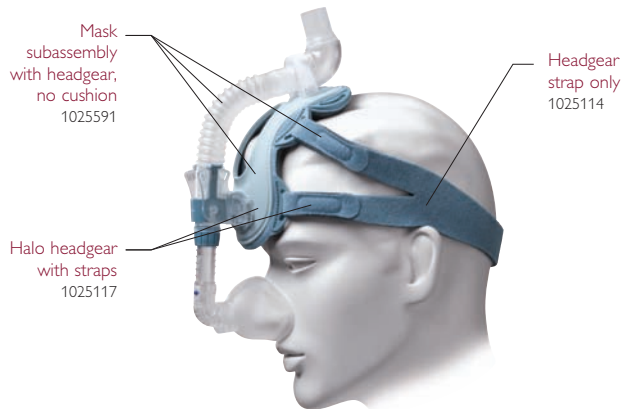

Pillows cushion
 1030502 (P)
 1030503 (S)
 1030504 (M)
 1030505 (L)




Simple cushion
 1021835 (P)
 1021830 (S)
 1021823 (M)
 1021821 (L)



Direct Seal
 1021802 (1)
 1021806 (2)
 1021807 (3)
 1021815 (4)
 1021816 (5)
 1021817 (6)

ComfortLite 2

The minimal contact alternative to nasal masks

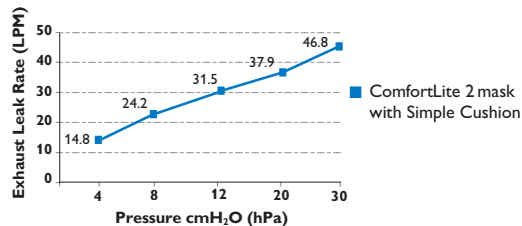


One cushion-type packs

	Simple cushion	Pillows	Direct Seal
S, M, L	1025171	1030501	1025170

Combo packs


Simple cushion and Pillows		Simple cushion and Direct Seal	
S and M	1030499	S,M and 4,5	1025168
M and L	1030500	M,L and 5,6	1025169

Minimal contact masks

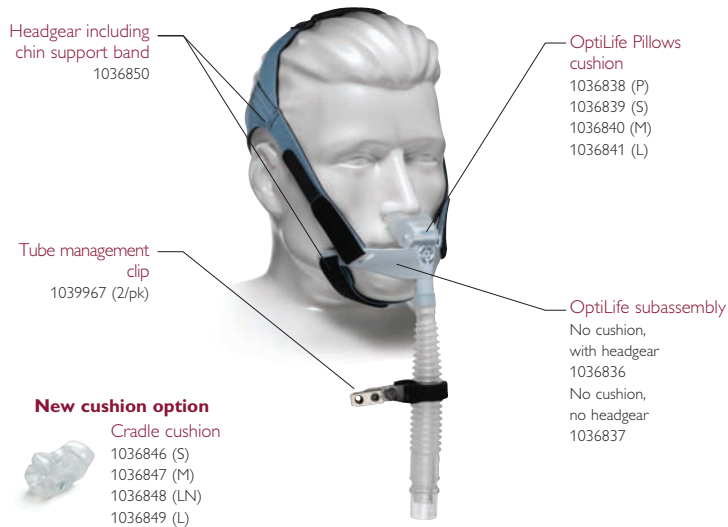

OptiLife

Maximum comfort in a 2-in-1 minimal contact mask solution

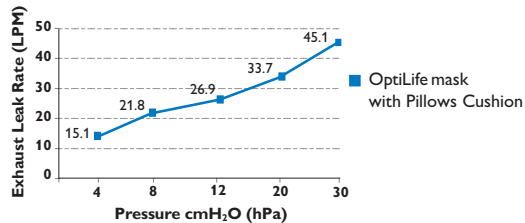

OptiLife FitPack with headgear and Pillows cushions

Each cushion size individually wrapped inside carrying bag

P, S, M, L

1036832

OptiLife 2-in-1 Pack with headgear

	Pillows and cradle	Pack content
S and M	1036833	Pillows P, S, M; Cradle S, M
M and L	1036834	Pillows M, L; Cradle M, L, LN

Minimal contact masks

Accessories

Headgear

Part number	Headgears
1033678	Premium headgear with EZ Peel tabs (fits perfectly to any type of nasal or full face mask)
1059257	Premium smaller size headgear
1040138	Blue Premium headgear with EZ Peel tabs (alternative headgear for ComfortGel Full mask)
1040844	Fusion headgear (alternative headgear for ComfortFusion mask)
1002801	Deluxe headgear (alternative headgear for ProfileLite and ComfortClassic masks)
1018755	Deluxe smaller size headgear
1014334	ComfortHeadgear (alternative headgear for ComfortFull2 mask)
1002285	SimpleStrap headgear
1050095	EasyLife headgear
1047975	FullLife headgear
1060807	FitLife headgear, small
1060808	FitLife headgear, large



1033678



1040138



1002801



1050095



1047975



1060807

Softcaps

Part number	Softcaps (only fit to ProfileLite, ComfortClassic, and Contour masks)	
Mesh softcap		
1001632	Mesh softcap - Child	P
1002806	Mesh softcap - Medium size	M
1002807	Mesh softcap - Large size	L
1001634	Mesh softcap - X-Large size	XL
White Lycra softcap		
1001624	White softcap - Child	P
1001625	White softcap - Medium size	M
1001626	White softcap - Large size	L
1001627	White softcap - X-Large size	XL
Blue Lycra softcap		
1001628	Blue softcap - Child	P
1001629	Blue softcap - Medium size	M
1001630	Blue softcap - Large size	L
1001631	Blue softcap - X-Large size	XL



Mesh softcap



White Lycra softcap



Blue Lycra softcap

Chin straps

Part number	Chin straps
1012911	Premium chin strap
1006085	Deluxe chin strap
302175	Chin strap



1012911



1006085



302175

Reusable circuits

Part number	Reusable circuits
622038	Reusable flexible tubing, grey (1,83 m)
1032907	Performance tubing, white (1,83 m)
1049270	Tubing insulator (for rainout and noise reduction)
1007800	High-temperature tubing, beige (1,83 m)
1006121	Reusable patient tubing with pressure line (1,83 m)



622038



1032907



1049270



1007800



1006121

Reusable valves and swivels

Part number	Reusable valves and swivels
302312	Plateau exhalation valve (PEV)
332113	Whisper Swivel 2
622148	Outlet swivel with pressure tubing
302418	Pressure valve



302312



332113



622148



302418

Other accessories

Part number	Other accessories
312710	O ₂ enrichment port
1014047	Bacteria filter
FC06566	Mouthpiece, angled, 22 mm
1004524	Mouthpiece, angled, 15 mm
452040	NG tube sealing pads (10/Pk)



312710



1014047



FC06566



452040

Accessories


3 steps to successful mask fitting

Size selection – step 1

Nasal mask sizing

Landmarks

- a. Sides of nostrils
- b. Bridge of nose zone
- c. Below the nose tip, above the lip

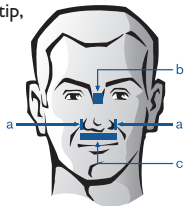

← Use the sizing gauge to select the right size →




Full face mask sizing

Landmarks

- a. Sides of mouth
- b. Bridge of nose zone
- c. Below the lower lip

Make sure the mouth is slightly open when checking size and the top of the cushion is not too high on the bridge of the nose area.

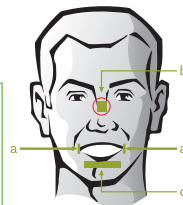

Mask fitting

Mask fitting – step 2

Common mistake
StabilitySelector adjusted
too high / bottom strap
too tight = leaks into
eyes, top lip pressure /
irritation




Common mistake
StabilitySelector adjusted
too low / bottom strap
too tight = leaks around
the mouth, forehead and
bridge nose irritation /
pressure points



How to adjust?

1. Click and slide the adjuster downwards
2. Loosen the bottom strap of headgear

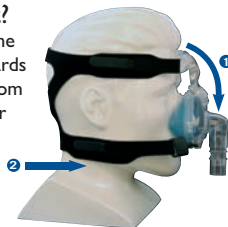


How to adjust?

1. Click and slide the adjuster upwards
2. Loosen top strap of headgear



One click on the StabilitySelector may make the difference.
Always move the StabilitySelector in the opposite direction
of the leak

OK

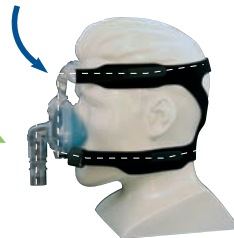

Correct position = mask parallel to the face
Do not overtighten the headgear

Final adjustments and cleaning instructions – step 3

Check the adjustments of the mask in lying position with the right pressure levels.



Cleaning instructions

Mask and tubing (wash daily)

- Hand wash the mask in warm water with mild dishwashing detergent.
- Rinse thoroughly and air dry completely before use.
- Wash face carefully before each use.
- Inspect the mask after cleaning; replace the mask and nasal interface cushion if damaged or worn.
- Rinse the tubing under shower and hang to dry.

Headgear (wash weekly)

- Handwash the headgear with mild detergent and warm water.
- Rinse thoroughly. Pat with a towel to absorb excess moisture, then hang to dry.

⚠ DO NOT use bleach, alcohol, any strong household cleaners, or cleaning solutions with alcohol, conditioners or moisturizers.

Additional fitting tips

- Check that the headgear is not overtightened: allow one finger/hand space between the face and each of the straps.
- Reposition the mask, if necessary, by gently pulling the cushion away from the nose and then back to the initial fit.



Mask fitting

Which device for which patient?

device for which patient?

Sleep therapy				Home N.I.V.			
Target patients							
	PR REMstar Plus	PR REMstar Pro C-Flex +	PR REMstar Auto with A-Flex	BiPAP autoSV Advanced	BiPAP Harmony	BiPAP Synchrony with AVAPS	Trilogy100
	OSA	OSA	OSA	Complex breathing pattern such as: CSR, CSA, mixed apnea, and complex apneas	Non-dependent chronic insufficiency respiratory diseases	Non-dependent chronic insufficiency respiratory diseases	Dependent chronic insufficiency respiratory diseases
	Fixed CPAP	Fixed CPAP	Fixed CPAP Auto CPAP	Setup possible as: Auto CPAP, BiPAP S and S/T, Auto CPAP + Servo Ventilation, BiPAP + Servo Ventilation	CPAP, S [spontaneous], S/T [spontaneous / timed]	CPAP, S, S/T, PC [pressure control], T [timed] + AVAPS	Pressure modes: CPAP, S, S/T, PC, PC-SIMV, T + AVAPS Volume modes: CV, AC, SIMV

 Humidifier available in option

Devices

Notes



Respironics and the name of Respironics products contained in this document are trademarks of Respironics, Inc. and its affiliates. All rights reserved.

©2010 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.

Broudy KB 02/10/10 MCI 4102856 PN 1067563

Philips Healthcare is part of Royal Philips Electronics

Respironics Asia Pacific
+65 6882 5282
Respironics Australia
+61 (2) 9666 4444
Respironics Europe, Middle East, Africa
+33 1 47 52 30 00
Respironics United Kingdom
+44 800 1300 845

Philips Respironics
1010 Murry Ridge Lane
Murrysville, PA 15668

Customer Service
+1 724 387 4000
800 345 6443 (toll free, US only)
www.philips.com/respironics